


Computation Constants for Thermodilution Cardiac Outputs

Edwards Swan-Ganz Catheter Model	Injectate Temperature								
	Cold Injectate (0-5°C)			Room Injectate (19-22°C)			Room Injectate (23-25°C)		
	10cc	5cc	3cc	10cc	5cc	3cc	10cc	5cc	3cc
096F6	0.547	0.259	0.144	0.582	0.280	0.161	0.608	0.305	0.180
131HF7	0.542	0.247	0.132	0.578	0.274	0.154	0.595	0.287	0.165
132F5	--	0.270	0.154	--	0.292	0.170	--	0.307	0.181
141HF7	0.542	0.247	0.132	0.578	0.274	0.154	0.595	0.287	0.165
143HTF7	0.554	0.259	--	0.587	0.286	--	0.599	0.291	--
C144H-7F/S144H-7F	0.547	0.252	0.134	0.579	0.277	0.156	0.597	0.295	0.169
C145H-6F/S145H-6F	0.547	0.252	0.134	0.579	0.277	0.156	0.597	0.295	0.169
151F7	0.542	0.247	0.132	0.578	0.274	0.154	0.595	0.287	0.165
831HF75	0.564	0.262	0.139	0.588	0.283	0.158	0.612	0.301	0.177
834HF75	0.564	0.257	0.143	0.582	0.277	0.156	0.607	0.294	0.170
931HF75	0.564	0.262	0.139	0.588	0.283	0.158	0.612	0.301	0.177
991HF8	0.568	0.268	0.147	--	--	--	0.616	0.302	0.176

Computation Constants Edwards CO-Set+ Closed Injectate Delivery System

Edwards Swan-Ganz Catheter Model	Injectate Temperature			
	Cold Injectate		Room Temperature	
	10cc (6-12°C)	5cc (8-16°C)	10cc (18-25°C)	5cc (18-25°C)
096F6	0.558	0.277	0.607	0.301
131HF7	0.561	0.259	0.608	0.301
132F5	--	0.285	--	0.307
141HF7	0.561	0.259	0.608	0.301
143HTF7	0.569	0.266	0.589	0.287
C144H-7F/S144H-7F	0.570	0.271	0.585	0.287
C145H-6F/S145H-6F	0.570	0.271	0.585	0.287
151F7	0.561	0.259	0.608	0.301
831HF75	0.578	0.272	0.592	0.290
834HF75	0.574	0.287	0.595	0.298
931HF75	0.578	0.272	0.592	0.290
991HF8	0.553	0.277 (8-12°C)	0.607	0.295

3cc injectate volume not recommended. All computation constants are for the proximal injectate lumen only. For any catheters not listed above call Edwards Lifesciences Technical Support at 800.822.9837.

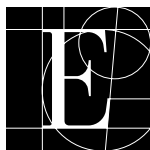

Edwards
LIFESCIENCES

Edwards Lifesciences devices placed on the European market meeting the essential requirements referred to in Article 3 of the Medical Device Directive 93/42/EEC bear the CE marking of conformity.

See package insert for full prescribing information.

Edwards Lifesciences, Edwards, the stylized E logo and CO-Set+ are trademarks of Edwards Lifesciences Corporation. Swan-Ganz and CO-Set are trademarks of Edwards Lifesciences Corporation and are registered in the U.S. Patent and Trademark Office.

This product is manufactured and sold under one or more of the following United States Patents: 4,329,993; 4,329,994; 4,407,304; 4,595,012; 4,651,751; 4,670,009; 5,387,199; and 5,354,272. Patent numbers in foreign countries available upon request.

Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.

© 2002 Edwards Lifesciences LLC
All rights reserved. 3195-2/02-CC



Edwards Lifesciences

Edwards Lifesciences LLC · Irvine, CA 92614 · Phone: 949.250.2500 · 800.424.3278 · www.edwards.com
Edwards Lifesciences (Canada) Inc. · Mississauga, Ontario · Canada L4Z 1S1 · Phone: 905.281.6700
Edwards Lifesciences S.A. · Switzerland · Phone: 41.21.823.4300
Edwards Lifesciences Japan · Chiyoda-ku Tokyo 102-0085 · Japan · Phone: 81-3-5213-5700